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Right picture
Shelf LifeSystem White Edition.

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Development of LifeSystem products is based on the following principles:

- Simple and intuitive use, independent of knowledge or people's verbal communication ability and ability to concentrate
- Appropriate size and accessibility for use; irrespective of height, posture or mobility of the user
- Flexibility in use due to adaptable, extendable and flexibly usable products

Made to function, LifeSystem offers decisive benefits:

- Adaptability to the user's body measurements
- Modular extendibility, even of existing products
- Flexible use in several functional areas

New 2007: White Edition

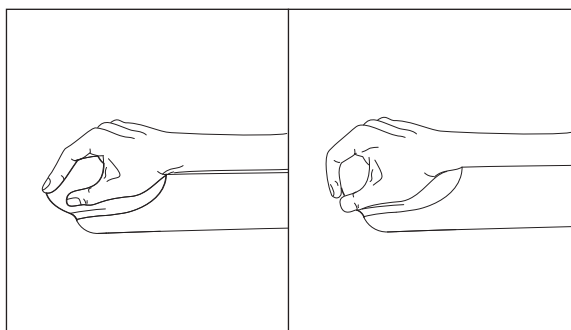
A new design option with a pure white surface.

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
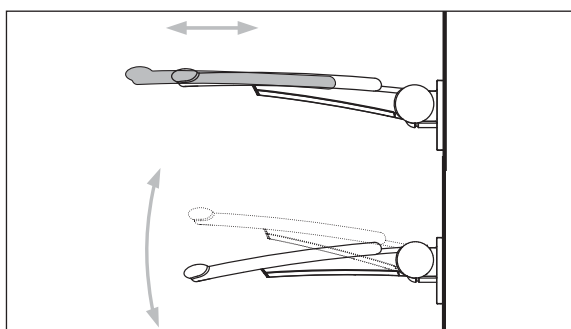
High degree of safety on gripping, straightening up, supporting, holding fast and turning were the priority when developing LifeSystem. The ergonomically shaped grip pad offers optimum hold during the sequences of movements arising.

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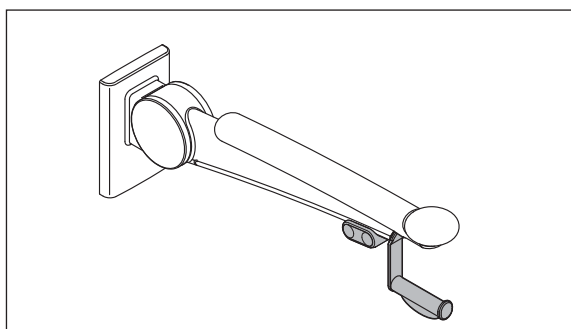The height and projection of the hinged support rail can be adjusted to the body measurements of the individual user. This results in increased and more comfort for residents, according to their height.

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
Upgradeability provides planning security

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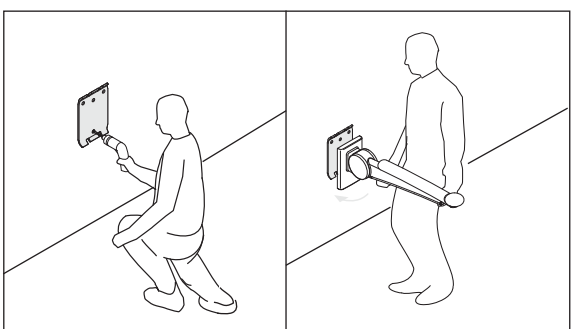
The hinged support rail is available as a basic model, which can be gradually supplemented with diverse upgrade kits. It can be optionally upgraded to include a WC paper holder, flushing and function keys with different technical versions and a backrest.

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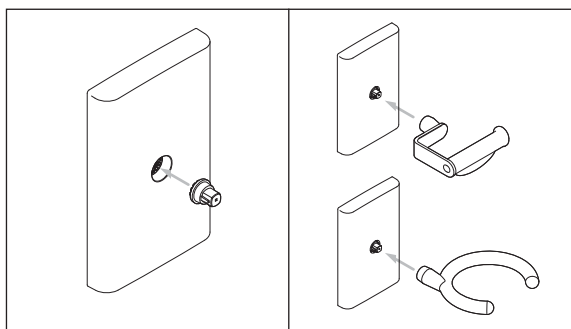The separate installation of wall-mounted fixing and product results in handling advantages. First the wall mounting plate is installed and then the product is hung in. This enables damage to the products on the construction site to be avoided.

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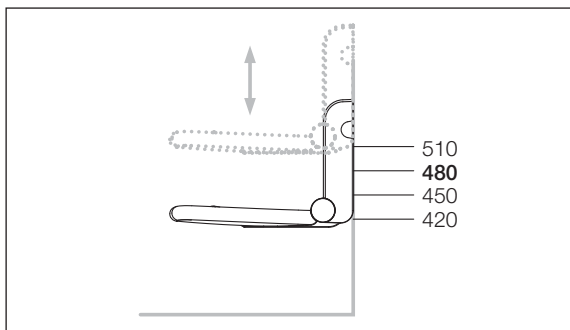
A selection of different accessories can be attached to the HEWI system interface, e.g. the walking aid holder, the WC paper holder and diverse hooks. This adapter can be used together with a rose, plate or wall mounting plate.

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- Secure gripping due to ergonomic grip pad
- Warm, agreeable to touch, plastic surfaces
- Height and projection adaptable to the user
- Simplified installation with separate wall mounting plate
- Upgradeability due to systematic product structure
- Highly robust, laterally stable joint
- Operable with minimal energy requirement

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
Convenient usability of the height adjustable shower seat

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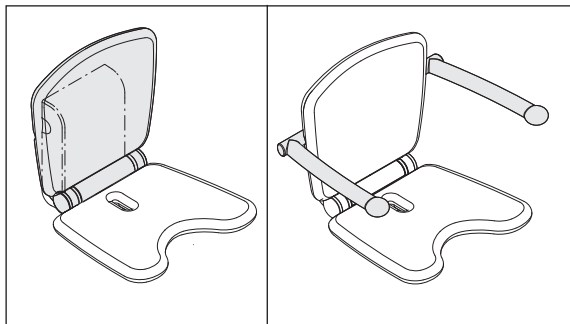A few simple hand moves by the care personnel are all that is required to adjust the height of the hinged seats. The height position is simply adapted to the patient/resident's height by pulling out the adjusting knob.

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
Upgradeability increases comfort and safety

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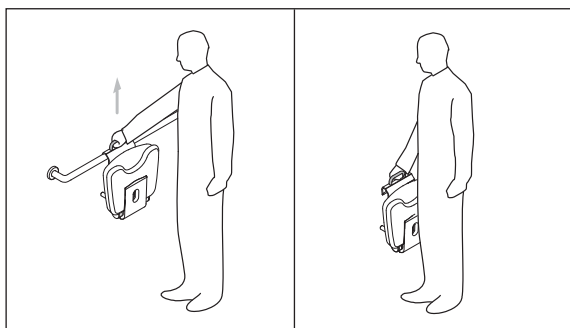
The hinged seat can be upgraded into a comfortable backrest. The individually fold-away armrests provide additional safety. In this way the costs do not exceed the planning basis of the initial fitout but offer future options.

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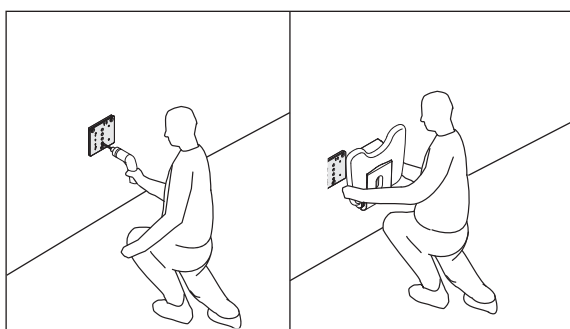The hanging seat is also available as a transportable hanging seat. This can be easily folded together by the care personnel and carried by the handle to other sanitary rooms.

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
The hinged seat wall mounting plate and product can also be installed separately by 1 person.

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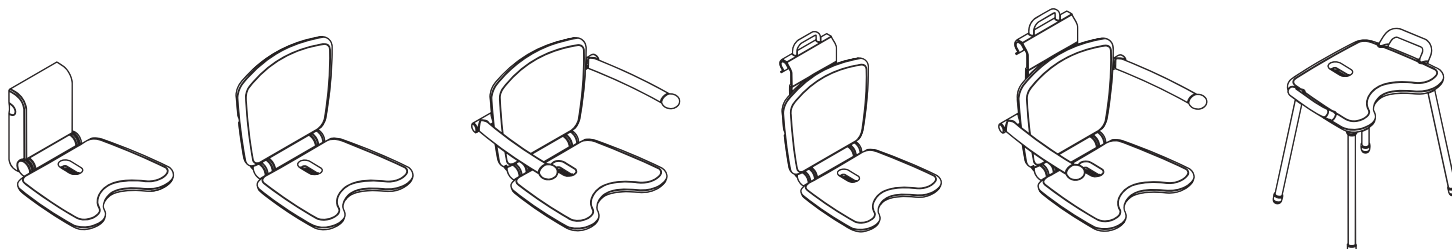- 4-level seat height adjustment possible
- Ergonomic seat surface with water runoff and hygiene opening
- Upgradable with larger backrest and armrests, thanks to the modular design
- Separate installation of wall mounting plate and product

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
Safety in handling patients/residents

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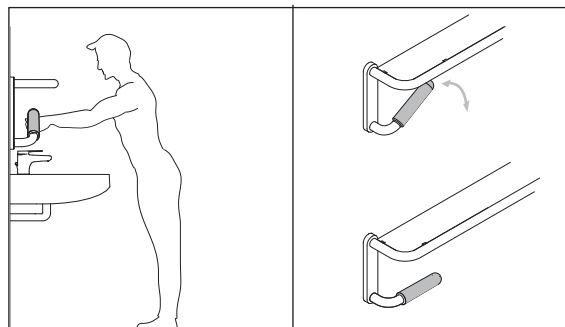The integrated support rails of the mirror shelf enable patients/residents to hold fast and steady themselves in an upright and/or seated position. This retains and promotes the user's independence and supports and eases the care work.

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
Good vision with continuous mirror

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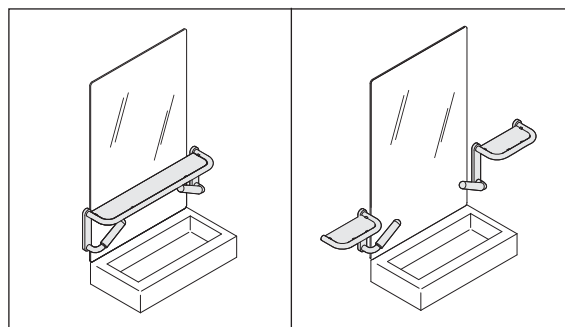The planning of continuous mirrors in barrier-free sanitary rooms is possible with the LifeSystem mirror shelves. With the divided mirror shelf it is also possible to look directly into the mirror from a seated position.

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
Mirror shelf divided

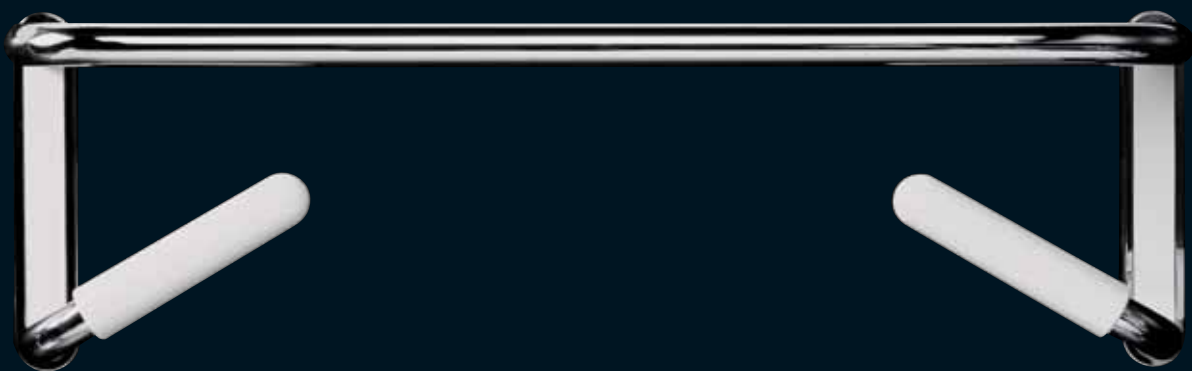- Continuous mirror solution possible
- Can be optionally used on the right or left
- High flexibility

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Mirror shelf


- Breakproof shelf with integrated grips for holding onto and for rising independently
- High stability, thanks to large attachment area
- Handle can be installed at 0° and 30° angle
- Continuous mirror solution possible

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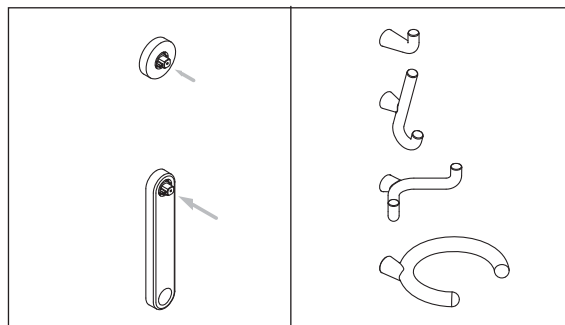
With the help of the HEWI system interface, different accessories can be optionally attached to a rose, plate or wall-mounting plate. This means diverse possible uses with a uniform type of fixing.

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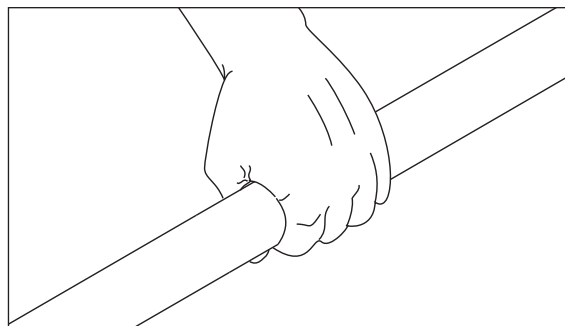Support rails in different configurations ensure a high degree of safety when gripping, straightening up, supporting and sitting down. The round tube system apart from the support rail is available in individual dimensions.

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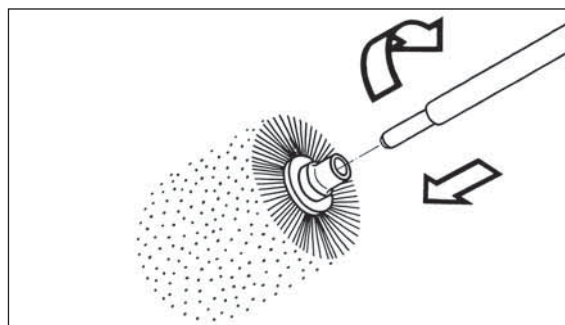
The brush head can be easily replaced by means of the bayonet fixing and therefore reduces costs. The brush heads are available in outer packagings of up to 25 brushes.

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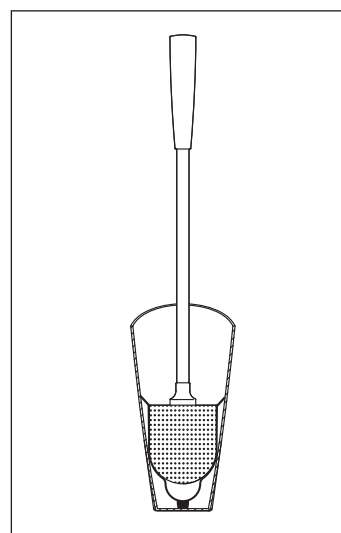
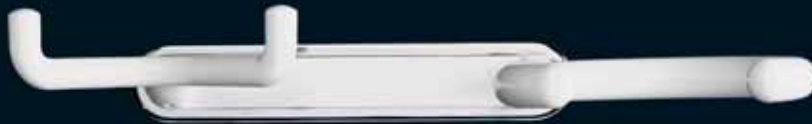
Long brush handle with ergonomically shaped grip at handle end. The brush is self-centring thanks to the conically shaped container. There is a reservoir for disinfectant in the bottom of the container.

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Basin area

Support rail, divided mirror shelf and shower stool.

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Toilet area

Hinged support rail and toilet brush set.

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Shower area

Hinged seat comfort, shower head holder set and curtain rail with shower curtain.

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czysta biel (99)

Application areas

With the pure white coated surface, the products can be easily integrated in the room.

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► **Softlink ...E**

By entering this code in the Softlink field on the HEWI Website you receive further product and/or service information.

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HEWI Kolory pastelowe

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Application areas

Hospitals, rehabilitation centres, nursing homes for the elderly, public buildings, private bathrooms – the LifeSystem collection includes products for the bathroom – washbasin, toilet, shower/bathtub.

HEWI Basic colours

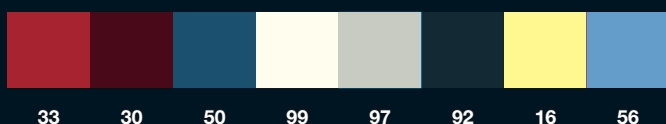
ruby red (33), burgundy (30), steel blue (50)

HEWI Neutral colours

pure white (99), light grey (97), anthracite grey (92)

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➤ **Softlink ...E**
By entering this code in the Softlink field on the HEWI Website you receive further product and/or service information.

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With an extensive product range available, it is difficult to make the right choice. To enable you to make a safe decision, you should note the following when planning barrier-free fitout in buildings:

Long-term viability of the investment

- High product quality = long life-cycle
- Maintenance free products
- CE marked medical products
- Corrosion-proof materials and fixings
- Possibility of cheap conversion
- Adaptable to user's body measurements
- Extendable product system = high planning security
- Easily integrated in architecture and care concepts
- System security: Compatibility with existing systems

Advantages for the user

- Comfortable handling
- Self-explanatory products
- Promotes mobility
- Promotes independence
- Product safety
- Functional design
- Easy and comfortable to grip surfaces
- Homely, pleasant ambience
- Joy in using the products

Advantages for the care personnel

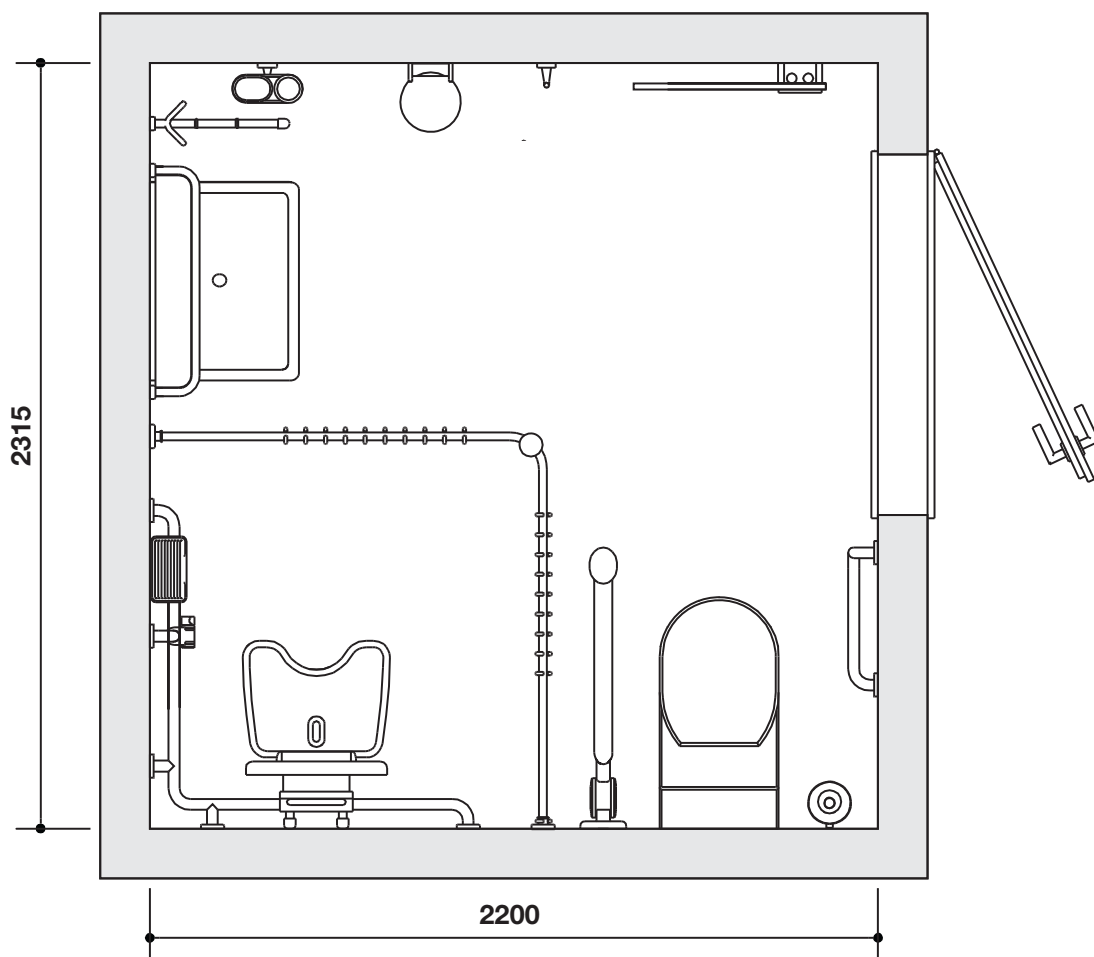
- Self-explanatory products
- Good usability
- Easy height and projection adjustability
- Relieves the care personnel
- Easy to clean
- Safety in handling patients/residents

Trade and installation

- Minimised stock levels due to modular product system
- Separate installation of wall mounting plate and product
- Installation aids
- 1-man installation possible
- Low installation times
- Easy handling (installation and upgradeability)
- No damage on the construction site
- Warranty according to ZVSHK

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Support from HEWI's planning & design service

With its planning & design service, HEWI offers a comprehensive service to assist you. We draw up an individual and directly implementable fitout proposal for you based on your measurements and floorplan sketch showing the position of the connections (on request, also supplied as 3D visualisation).

We not only name the suitable components of the HEWI product range, but also provide recommendations for combination with other manufacturers' sanitary products.

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Using the following HEWI services simplifies your daily work for the design and planning of a building.

Consultation:

- Comprehensive advice in all phases of planning through the entire spectrum of HEWI products and services
- Support in invitations to tender
- Samples
- Producing individual solutions
- Support for presentations to project owners and investors
- Use of teams of HEWI specialists to deal with the themes of barrierfree living, electronic locking systems, signage systems, together with individualised solutions for handrails
- Information on legal regulations

Technical presentations:

- Product catalogue on the Internet and on the HEWI Infoware CD
- Sample blocks and colour swatches
- Provision of materials for showboards
- Exhibition systems

Deutschland

HEWI Heinrich Wilke GmbH
Postfach 1260
D-34442 Bad Arolsen
Telefon: +49 5691 82-0
Telefax: +49 5691 82-319
info@hewi.de
www.hewi.de

HEWI Polska

HEWI Heinrich Wilke GmbH
ul. Lelka 26
02-812 Warszawa
tel. 0048 22 330 00 51
fax 0048 22 330 00 50
info@hewi.pl
www.hewi.pl

www.hewi.com